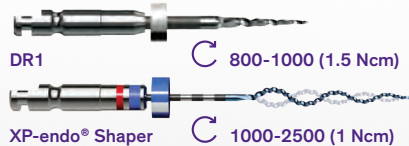
INFO

**EN**

Create a start in the coronal portion of the gutta percha (3-5mm) using DR1.

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Place one drop of solvent in the coronal space created and wait for at least 1 minute.

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Place the tip of XP-endo Shaper into the prepared space in the gutta percha, disengage slightly and start the motor at 1000-2500 rpm.

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Use pecking motion until the XP-endo Shaper's tip engages the gutta percha. Then use light pressure to help advance the XP-endo Shaper down the canal until working length.

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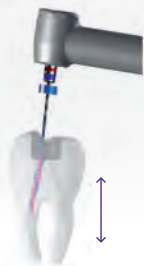
XP-endo® Finisher R  800-1000 (1 Ncm)

## EN

Allow the XP-endo Shaper to gently corkscrew around the gutta percha and tease out large strands of gutta percha using long gentle strokes.

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After the removal of the gutta percha, measure the working length (WL), irrigate and fill the pulp chamber and canals with NaOCl. Use the XP-endo Shaper for 10-15 additional long gentle strokes to WL.

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Use the XP-endo Finisher R for at least 1 minute (longer, if debris is still coming out of the canal), using slow and gentle longitudinal movements to contact the full length of the canal.

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Use copious NaOCl irrigation and suction to remove the gutta percha and sealer tags prior to complete the treatment.

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